

1524-30  
LUMBAR HOLDER

102539  
IV BAG 100mL STERILE WATER

1324-62-3  
L1-SACRUM WITH  
LAMINECTOMY

1324-62-1 LATEX DURA TUBING WITH FITTINGS  
(1485-99 LATEX TUBING ONLY)

1485-99-1  
IV SET GRAVITY DRIP LINE WITH FITTINGS

## Replacement parts

Kit Options

\*2013 pricing

Part #	Description	Price
1324-62	Complete kit including lumbar holder, lumbar vertebrae, dura tubing with release fittings, IV gravity drip line and IV bag	\$300.20
1324-62-2	Kit includes lumbar holder, lumbar vertebrae, dura tubing with release fittings, and IV gravity drip line (no IV bag)	\$276.70

Replacement Parts

\*2013 pricing

Part #	Description	Price
1524-30	Lumbar Holder	\$120
1324-62-1	Dura tubing with release fittings	\$64.30
1324-62-3	L1-SAC with laminectomy & 3/4" hole in sacrum	\$80.10
1485-99-1	IV set, gravity drip, 10 drops/mL	\$12
102539	iv bag 100ml sterile water	\$16
1485-99	Latex tube only	\$1.10



**Installing/replacing dura component: Feed the blue end of the dura replacement unit up and through the large hole in the sacrum. Slide the blue tip all the way to the L1. Correct positioning shown below.**






**Take the spike end of the IV tube set and connect to the sterile solution water bag.**




**Take the luer lock end of the IV tube set and connect to the male end of the 1324-62-1 dura tubing unit by twisting it on.**



**Open the blue valve at the top of the dura. Open the water line and allow the air to escape thru the end of the dura tubing. Once the dura is filled with water (no bubbles), close the blue valve and leave the water flow on the IV solution open.**

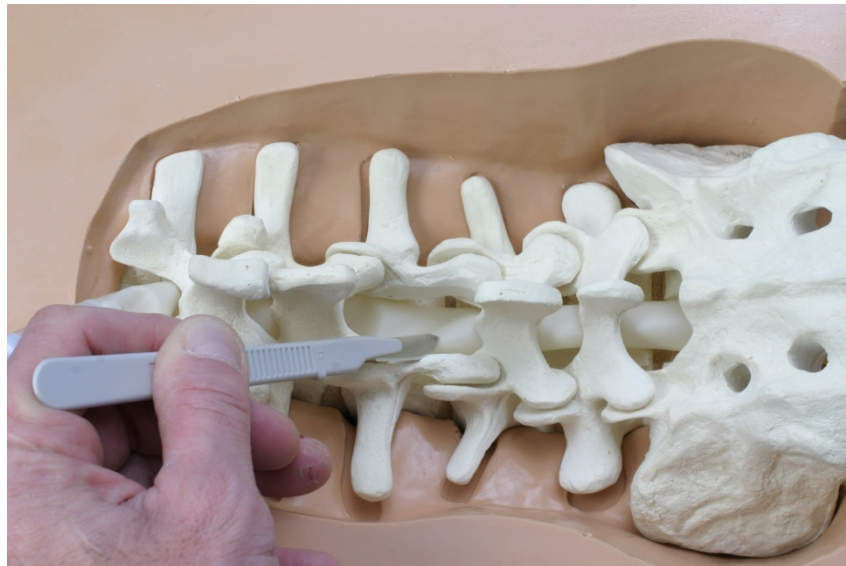


**Create a 10mm “nick” incision at the laminectomy site. Reduce water flow and proceed with dura repair using an Ethicon Prolene 6-0 suture or similar.**

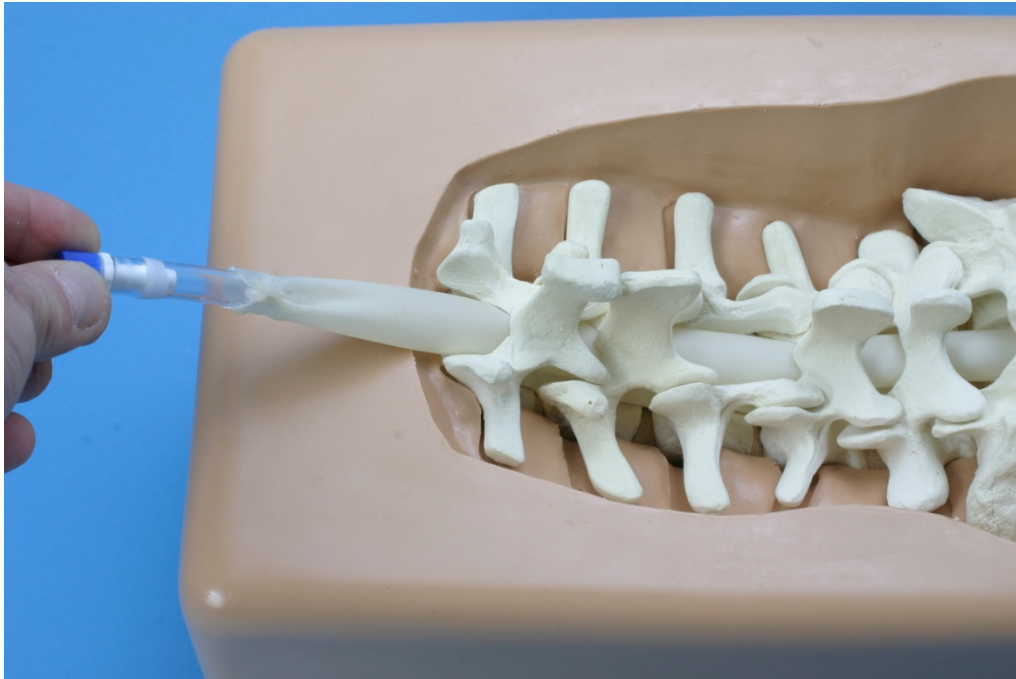


**By advancing the dura, 2 to 3 separate procedures can be performed on one replacement model.**

**Disconnecting/Replacing Dura Unit:**

**Turn off the drip line at the bag by rolling the white wheel down.**

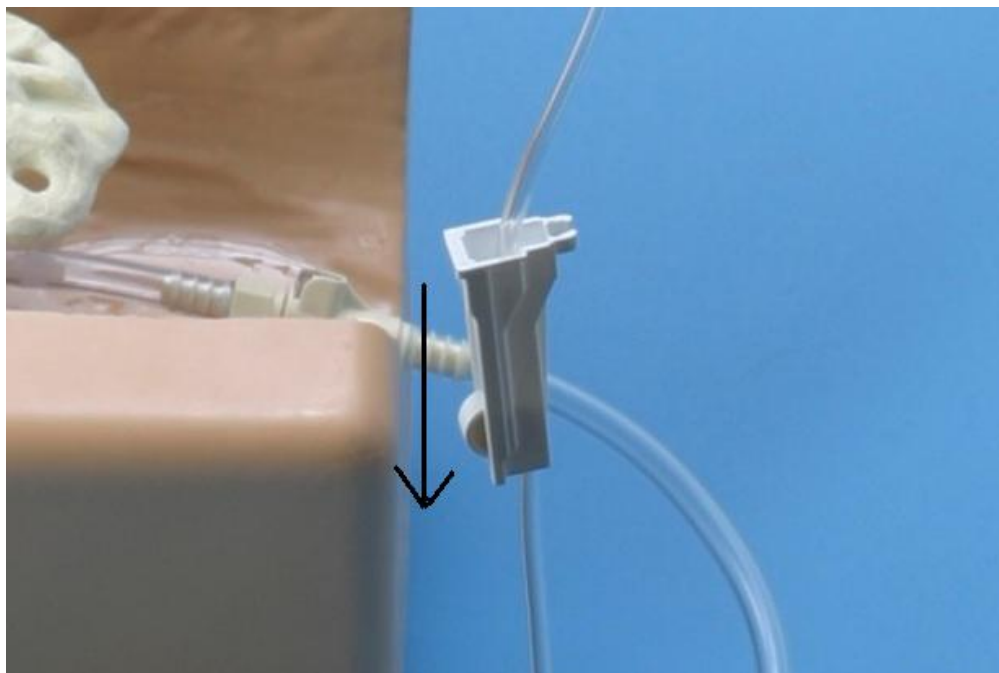

**Disconnect the luer lock at the dura unit by twisting counter-clockwise. Note the check valve is attached to the dura side. This will prevent water from leaking.**

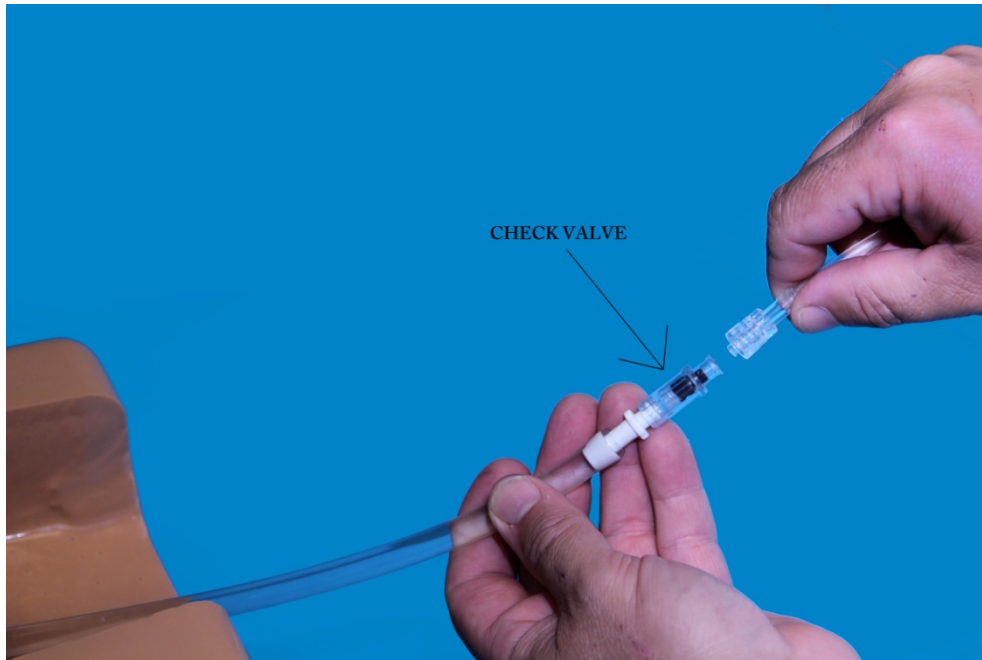

**Refer to the start of this set-up manual for attaching the new dura unit.**

### **Model revision update 9/2012:**

**We have made an update to the attachment fittings for the IV line to the dura unit. If you purchased your unit before September 2012, you will need to remove your quick release fitting from the IV bag. All that is needed is the luer lock at the end of the IV bag. This will attach directly to the check valve on all new dura units. If you have any questions, feel free to contact our customer service at 206-463-5551.**