



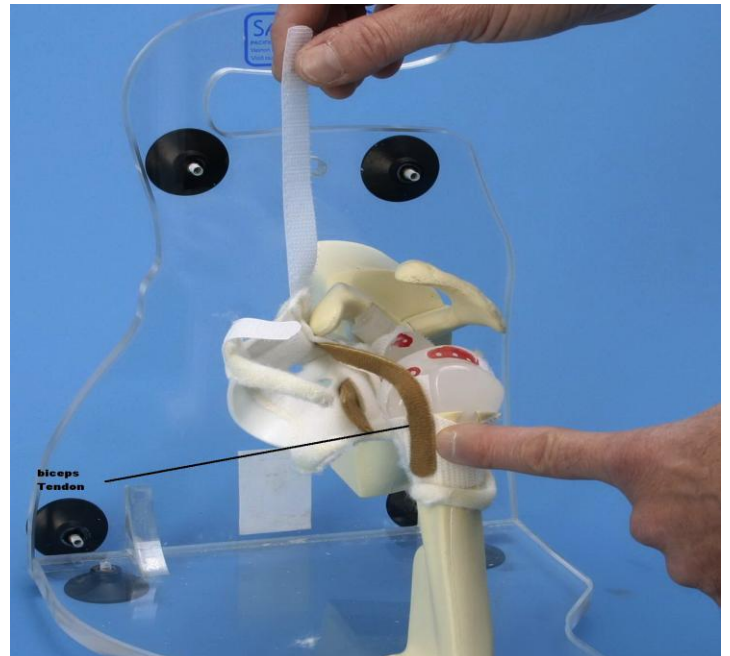

One

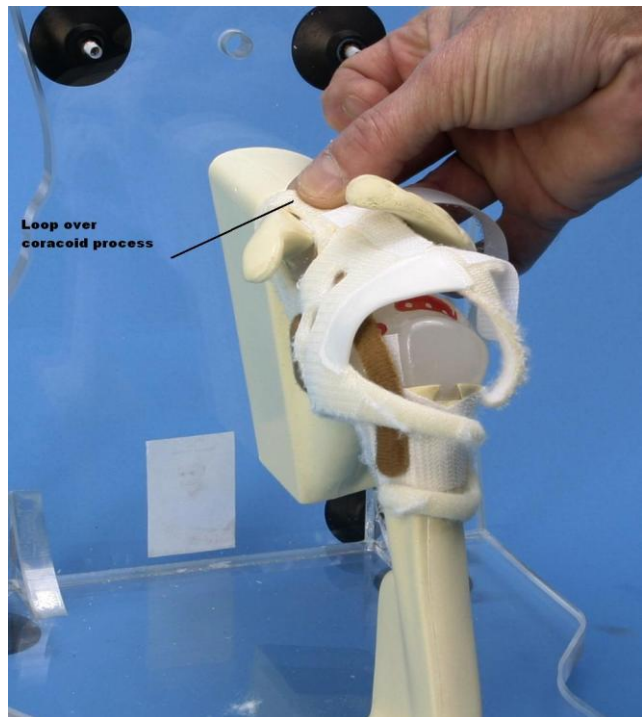

Two



Three




Four

Image one Is the entire piece assembled.

Image two. Start by getting the biceps tendon oriented then the base of the labrum on the inferior side of the glenoid.

Image three. Take the loop of the labrum and place it over the Corcoid process. This loop is the only one on the piece so once that is in place the overall orientation should be good.

Image four. There is a white rubber piece that should be positioned on the humeral head along the groove just off the articular surface. (Positioned for suture anchors) Then there will be 4 narrow bands of material trailing down and those wrap along the base of the humeral shaft. There is some flexibility of positioning of these as they act more as stabilizers.

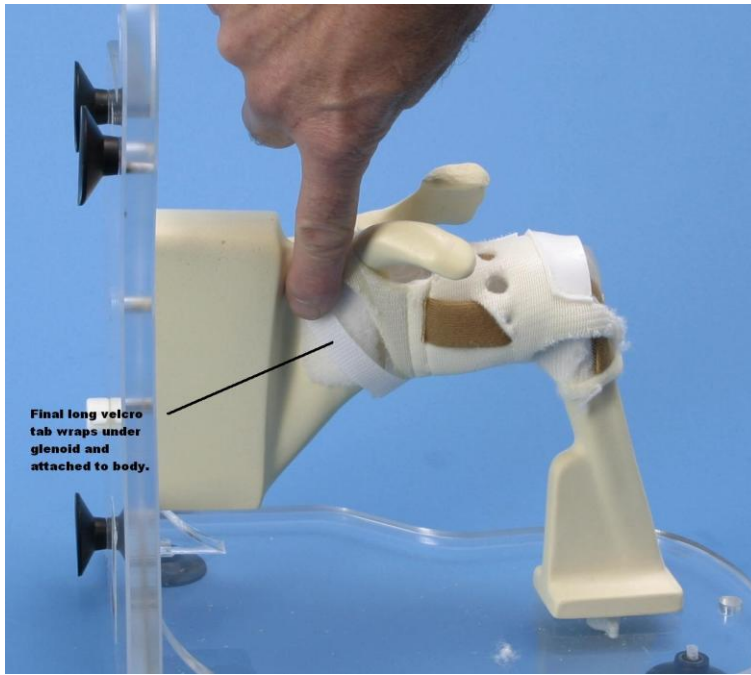

Five

Image five. The final long Velcro tab wraps under the glenoid body and attaches to the base of the humerus.