

Bone insert removal:

- 1 Remove the model from the bone clamp.
- 2 Remove the Velcro strip from the posterior section of the soft tissue sleeve.
- 3 Retract the soft tissue to gain access to the insert.
- 4 Remove the proximal humerus first.



Bone insert replacement:

- 1 Retract the soft tissue slit with fingers.
- 2 Install insert into sleeve, ulna/radius first.
- 3 Fully seat within cavity.
- 4 Replace Velcro strip and reinstall model into bone clamp.




Should you have any questions, please contact:

SAWBONES
 10221 SW 188th Street
 Vashon, WA 98070
 (206) 463-5551
 info@sawbones.com

SAWBONES EUROPE AB
 Krossverksgatan 3
 216 16 Malmö
 Sweden
 +46 40 650 70 00
 info@sawbones.se

www.sawbones.com

**Elbow Arthroscopy Model
 Item # 1411**

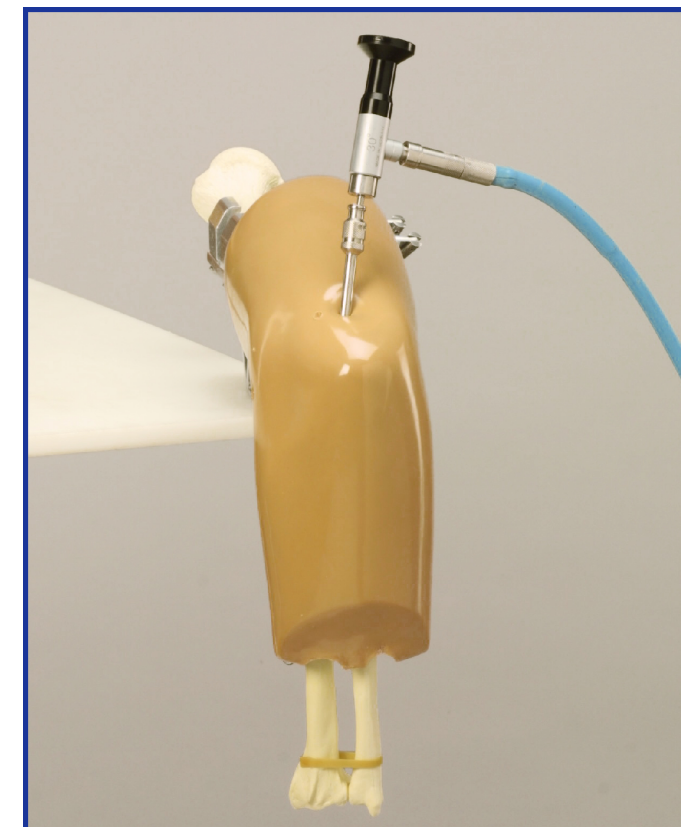


Thank you for purchasing the Sawbones elbow arthroscopy model. This arthroscopy model may be used for diagnostic and visualization skills including triangulation and pathology identification. Loose body removal may also be performed. The model may be manipulated in pronation and supination. Custom inserts can be developed for specific applications upon request. With a little care and maintenance your new model should provide you with many years of service.

This instructional brochure will take you through:

- * Components and anatomical structures
- * Set up instructions
- * Insert removal and replacement instructions
- * Care and maintenance
- * Available replacement parts



COMPONENTS & ANATOMY

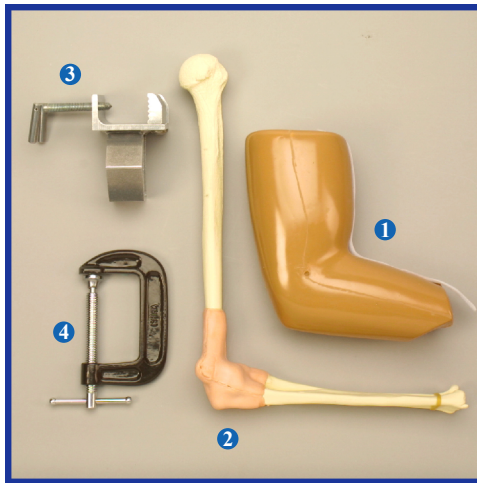


Components:

- 1 Soft tissue elbow
- 2 Elbow bone insert
- 3 Bone clamp
- 4 C-clamp

Anatomical structures:

- * Cartilage erosion of the capitellum
- * Radial head fracture
- * Multiple loose bodies

SET UP

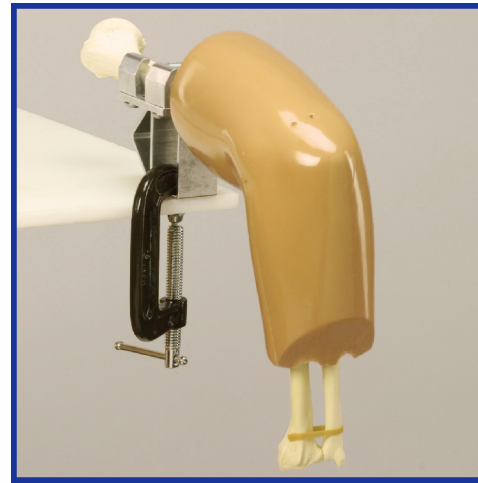


- 1 Use the bone and C-clamp combination to attach the model to the table as shown.
- 2 You may position the model in a variety of ways.

REPLACEMENT PARTS



Elbow joint
Item # 1411-2

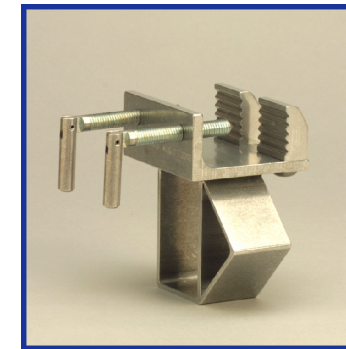

Bone clamp
Item # 1600



C-clamp
Item # 1601

CARE & MAINTENANCE

Your model can be wiped down with any mild cleaner. Use only the dry erase marker on the model. Standard ink can not be removed. Store in a dry, dark place out of direct contact with sunlight.